

Impression Material Accessories Technique

5th Hand™ Retractor



- 1 After choosing the correct size of the 5th Hand for your patient, place the foam pads included in your 5th Hand kit, around retractor for more comfort.



- 2 Place retractor in patient's mouth, and pull gently outward to create space for the compression tray.




- 3 Place loaded impression tray in mouth. Once seated and before set, remove the 5th Hand by pulling it straight out.

Dam it!™ Membrane



- 1 Remove membrane from dispenser by pulling the printed membrane up and out.

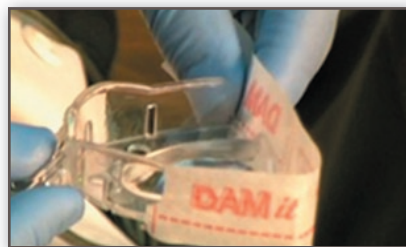

- 2 Place self adhesive membrane around the posterior of the tray, being careful to keep the dotted line along the floor of the tray.



- 3 Cut slits in membrane at the opening. Fold and tuck the excess membrane against the impression tray.



- 4 For a quadrant tray, use one repeat, 1" to 2" of membrane on each side.



- 5 Fold the membrane against the tray for a water tight seal.



- 6 Cut or fold against the tray. After folding, load the tray, seat over teeth.

